

*No. 1 choice  
of hospitals  
& mothers*

# Swing™

- EN** Instructions for use
- RU** Инструкция по применению
- HU** Használati útmutató
- CS** Návod na použití
- TR** Kullanım talimatları
- PL** Instrukcja obsługi

**EN** Parts List  
**RU** Список деталей  
**HU** Alkatrészek listája  
**CS** Seznam dílů  
**TR** Parça Listesi  
**PL** Lista części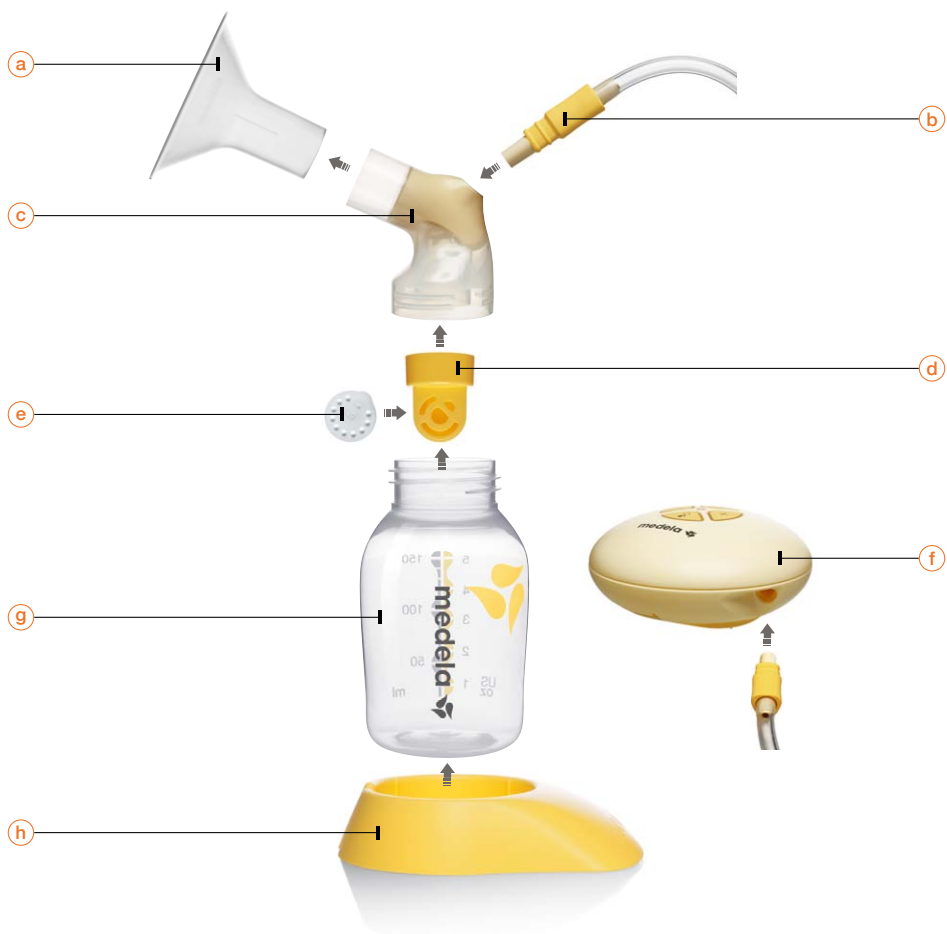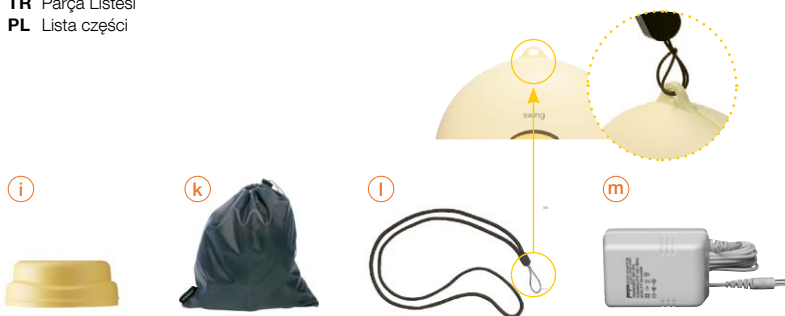

**EN** Parts List  
**RU** Teileliste  
**HU** Alkatrészek listája  
**CS** Seznam dílů  
**TR** Parça Listesi  
**PL** PersonalFit™ Lejek dzielony

**a** 810.7084

**EN** PersonalFit™ Breastshield 24mm  
**RU** Воронка PersonalFit™  
**HU** PersonalFit™ szívófej  
**CS** Prsní nástavec PersonalFit™ (24 mm)  
**TR** PersonalFit™ Meme Hunisi 24 mm  
**PL** PersonalFit™ Lejek dzielony

**b** 800.0836

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TR  
44–53

PL  
54–63

Thank you for choosing the Swing Breastpump. Breastmilk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close co-operation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breast pumps that perfectly meet breastfeeding mothers' needs. For mothers who insist on the best for their babies and themselves. Because you care.

## Why Swing™?

Swing is the best solution, if you want to nourish your baby with your milk and remain flexible and mobile. The innovative 2-Phase technology enables you to pump efficiently and effectively. Moreover Swing is easy to handle, light to transport, perfect for the dynamic mother.

### What is 2-Phase Expression®?

Medela has been the first company worldwide to deeply research babies and their feeding behaviour. The results have led to the development of the 2-Phase-Expression-Technology. 2-Phase-Expression is based on the natural sucking rhythm of the baby: Short, stimulating movements (**Stimulation**) are followed by longer, more intensive ones (**Expression**), ensuring an optimal milkflow. Thus, more milk can be expressed in a shorter time – and more comfortably.

## Indications for use

Swing is an electric or battery powered breastpump for supporting breastfeeding mothers. Swing is ideal:

- I To help overcome mother or baby breastfeeding problems
  - To relieve sore and cracked nipples
  - To bring out flat or inverted nipples

# Important safeguards

When using electrical devices, basic safety precautions should always be followed, including the following: Read all instructions before use.

## Product

- | Keep appliance dry and do not place in direct sunlight.
- | Do not get the product wet.
- | Do not place in or drop motor unit into water or other liquid.
- | Do not spray or pour liquid directly onto the pump.
- | Check pump kit components for wear or damage.
- | This is a single user product. Use by more than one person may present a health risk.
- | There are no user-serviceable parts inside the Swing.
- | Do not attempt to repair the Swing Breastpump yourself.

## Electrical parts

- | Only use the mains adapter that comes with the Swing Breastpump.
- | Make sure the voltage of the mains adapter is compatible with the power source.
- | Plug the mains adapter into the Swing Breastpump first and then into the wall socket.
- | Always unplug electrical devices immediately after using.
- | Separation from the mains is only assured through the disconnection of the mains adapter from the wall socket outlet.
- | Keep the mains adapter away from heated surfaces.
- | Do not reach for any electrical device if it has fallen into water. Unplug immediately.

## Use


- | Use the product only for its intended use as described in this instruction manual.
- | Pump only with the device in an upright position.
- | Use Medela original accessories only.
- | Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- | The product should never be left unattended when plugged into a power source.
- | Do not use outdoors.
- | Do not use while bathing or showering.
- | Do not use while sleeping or overly drowsy.
- | Supervision is necessary when this product is used in the vicinity of children.
- | If problems or pain occur, consult your lactation consultant or physician.
- | Equipment not suitable for use in the presence of explosive atmospheres.
- | Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

- | Portable and mobile radio frequency communications equipment can affect the Swing Breastpump.
- | The Swing Breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.

SAVE THIS INSTRUCTION MANUAL.

# How to clean Swing pump set

Please follow these handling and cleaning instructions. Use drinking water quality only. Disassemble all parts prior to cleaning.

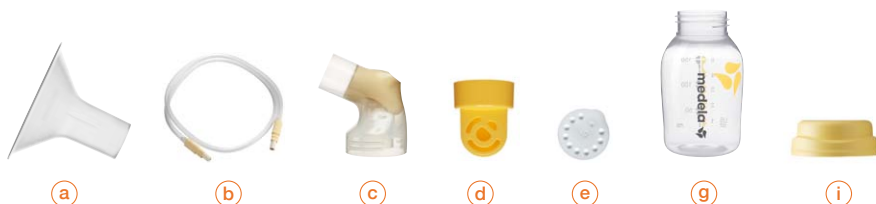Part	a	b	c	d	e	g	i
<b>Clean prior to first and once daily</b>							
Firstly rinse with hot water by hand, second wash with warm soapy water, finally rinse in clean hot water. Medela also recommends	✓	✓	✓	✓	✓	✓	✓
covering with cold water in a saucepan and boiling out for 5 min.	✓	✓	✓	✓	✓	✓	✓
or washing in top rack of dishwasher	✓	✓	✓	✓	✓	✓	✓
or with Quick Clean Microwave Bag	✓	–	✓	✓	✓	✓	✓
or with b-well Steam Steriliser	✓	–	✓	✓	✓	✓	✓
<b>Clean immediately after each use</b>							
Firstly rinse with hot water by hand, second wash with warm soapy water, finally rinse in clean hot water.	✓	✓	✓	✓	✓	✓	✓

**Note:** Food colourings may discolour components which will not affect performance. Medela assumes no liability for improper handling. You can find more information at: [www.medela.com](http://www.medela.com)

# How to prepare your breastpump

## Electrical operation

Firstly, plug the adapter cable into the pump, then connect to the power socket.

## Battery operation

Open the cover of the battery compartment in the direction of the arrow.

Insert the AA Mignon LR6 batteries and close it again.

**Note:** The batteries do not need to be removed if the main adapter is used for pumping.

# How to assemble your Swing pump kit

Push the breastshield onto the connector,

then place the membrane on the valve head

and push the valve onto the connector.

**Note:** ensure sidewise position of valve head.

Screw the connector onto the bottle.


Insert the tubing into the connector.

Insert the other end of the tubing into the tubing port on the motor unit.

# How to use your Swing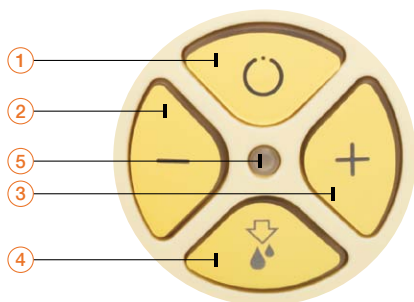

- 1 On/Off button
- 2 Vacuum decrease button (-)
- 3 Vacuum increase button (+)
- 4 Let-down button
- 5 LED  
lights up → On  
does not light up → Off  
regular flashing → Stimulation Phase  
light constantly on → Expression Phase  
rapid flashing → Min./Max.


**Note:** If the pump has been running for 30 minutes without any manipulations, the Swing will automatically switch off.

Centre breastshield over your nipple.

Switch on the breastpump by pressing 1. Now the Stimulation Phase starts.

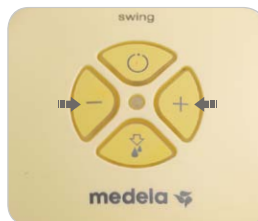Adjust the vacuum strength by pressing 2/3.

Once your milk starts to flow, press 4 to switch to the Expression Phase. After two minutes the pump will automatically switch from the Stimulation to the Expression Phase.

# How to store/thaw breastmilk

Medela recommends using the following guidelines for storing and thawing breastmilk. These apply to healthy, full-term babies.

	Room temperature	Refrigerator	Refrigerator freezer compartment	Freezer
<b>Freshly expressed breastmilk</b>	Do not store at room temperature	3-5 days at 4°C	6 months at approx. -16°C	12 months at approx. -18°C
<b>Thawed breastmilk (previously frozen)</b>	Do not store	10 hours	Do not refreeze after thawing!	

- I Never fill bottles/bags more than  $\frac{3}{4}$  full to allow space for possible expansion.
- I Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

# Troubleshooting

Problem	Solution
<b>If the motor is not running</b>	Check presence of mains current. Check the position of the batteries in the battery compartment.
<b>If the pump suction is too weak or there is no suction</b>	<p>Verify the valve head and membrane are clean and not damaged. Make sure the white membrane is laying flat against the valve head.</p> <p>Ensure sideways position of valve.</p> <p>Check all connections of the kit and breastpump.</p> <p>Check the ends of the tubing are fit securely into the back of the breastshield and into the port on the breastpump.</p> <p>Take care not to kink tubing.</p> <p>Ensure the breastshield forms a complete seal around the breast.</p>
<b>If condensation appears in the tubing</b>	Rinse out the tubing. Shake out water droplets and hang tubing to air-dry.
<b>If milk gets into the tubing</b>	<p>Turn off the pump and unplug from power source.</p> <p>Remove and clean the tubing.</p> <p>Shake out water droplets and hang to air-dry.</p>
<b>If milk accidentally gets into the pump</b>	<p>Turn off the pump and unplug from power source.</p> <ol style="list-style-type: none"> <li>1. Open the battery cover, remove the small, black foam piece from bottom of pump. Rinse the foam piece with clear, lukewarm water and squeeze out well. Allow to dry thoroughly.</li> <li>2. Pour about 2 dl of warm soapy water into a container.</li> <li>3. Connect the tubing and the power cable onto the pump.</li> <li>4. Hold the device with the control button upwards over a dry flushing bath and switch on. Press 4 and by pressing 2 multiple times/twice, ensure that the lowest vacuum level is set.</li> <li>5. Hold the end of the tubing that is not connected to the pump in the container with the soapy water.</li> <li>6. The water is pumped through the tubing into the device. Allow it to pump until all the water has been flushed through.</li> <li>7. Then leave the device switched on for 3 minutes, until the pump is completely dry.</li> <li>8. As soon as the expanded-plastic liner is dry, put it in place again and close the battery cover.</li> <li>9. If necessary, clean pump kit according to the instructions „Prior to first use and once daily“.</li> </ol>

# Warranty

In the "International Warranty" booklet you find all information about the warranty.

# Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects on the environment and on public health.

## ....one more question...

### **Am I using the right size breastshield?**

... Every mother is different and needs her personal size, therefore Medela offers five different sizes of PersonalFit breastshield. Answer a few short questions at [www.medela.com/IW/en/breastfeeding/good-to-know/right-size-of-breastshield.html](http://www.medela.com/IW/en/breastfeeding/good-to-know/right-size-of-breastshield.html) to find out if the available size fits.

Visit our website at [www.medela.com](http://www.medela.com). Here you will find answers to all your questions.

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IEC 60601-1-2, Table 201

## Electromagnetic Emissions

The Swing is intended for use in the electromagnetic environment specified below.  
The customer or the user of the Swing should assure that it is used in such environment.

Emission Tests	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Swing uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Swing is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 60000-3-3	Complies	

**Warning** – The use of accessories and cables other than those specified may result in increased emissions or decreased immunity of the Swing breastpump.

The Swing breastpump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Swing breastpump should be observed to verify normal operation in the configuration in which it will be used.

The Swing breastpump was found to have no essential performance by a risk analysis and thus, was not tested for immunity to electromagnetic disturbances (IEC60601-1-2:2001, 6.8.3.201 i)

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SWING BREASTPUMP  
WITH RESPECT TO ELECTRIC SHOCK FIRE, AND  
MECHANICAL HAZARDS ONLY IN ACCORDANCE  
WITH UL60601-1/CAN/CSA C22.2 No. 601.1 22PL

[www.medela.com](http://www.medela.com)

Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
[www.medela.com](http://www.medela.com)

#### **INTERNATIONAL SALES**

Medela AG  
Lättichstrasse 4b  
6341 Baar  
Switzerland  
Phone +41 (0)41 769 51 51  
Fax +41 (0)41 769 51 00  
[ism@medela.ch](mailto:ism@medela.ch)  
[www.medela.com](http://www.medela.com)

#### **POLAND**

Medela Poland Sp. z o.o.  
Ul. Lewinowska 8  
03-684 Warszawa  
Poland  
Phone +48 22 864 38 65  
Fax +48 22 865 12 50  
[biuro@medela.pl](mailto:biuro@medela.pl)  
[www.medela.pl](http://www.medela.pl)

#### **RUSSIA**

OOO Medela  
Vavilova str. 97, entrance 3  
117335 Moscow  
Russia  
Phone +7 (495) 980 6194  
Fax +7 (495) 980 6194  
[info@medela-russia.ru](mailto:info@medela-russia.ru)  
[www.medela-russia.ru](http://www.medela-russia.ru)